

Camera assembly and settings:

DermoGenius II + Photoadapter+ digital camera with special adaptation



dermo
scan^{GmbH}

Please look carefully to the following instructions before you activate camera and software - if any read the short description. When needed, contact

DermoScan GmbH


Prüfeninger Schloßstraße 2

D-93051 Regensburg

Telefon-Hotline: +49 (0) 941 / 99 22 60 65

Fax-Hotline: +49 (0) 941 / 99 22 22 35

Camera assembly:

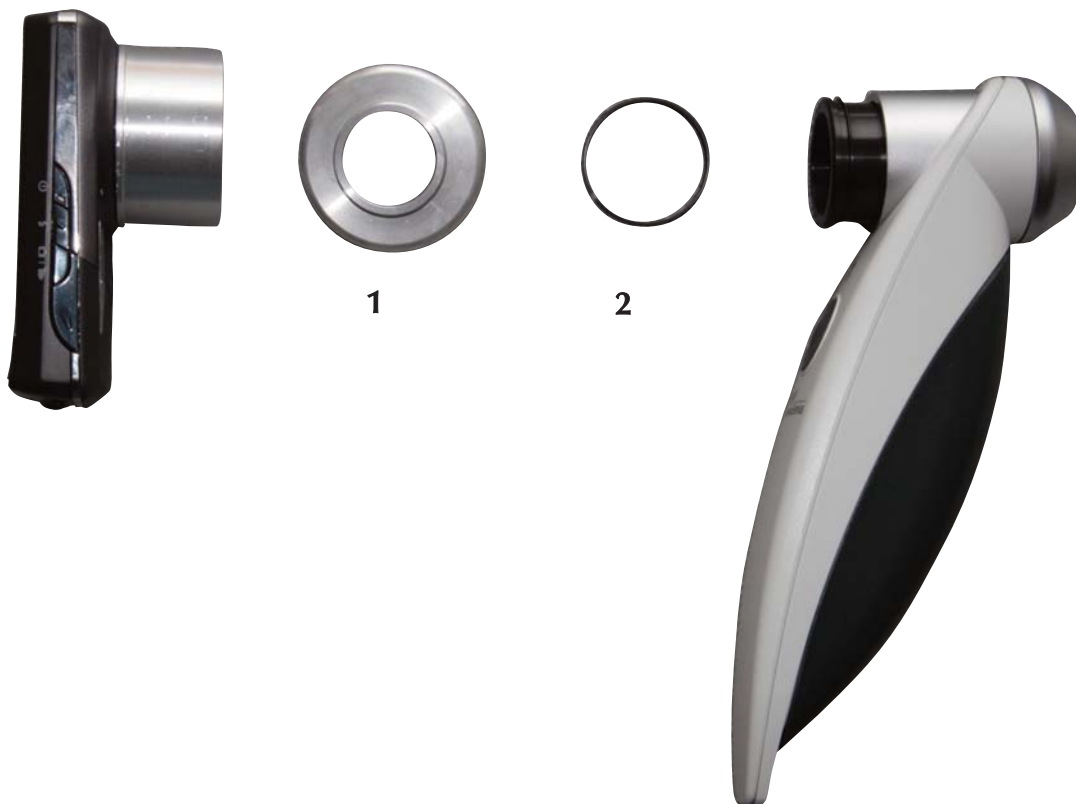

- ◆ Screw photoadapter (1) for DermoGenius II onto the special adaptation on the camera
- ◆ Unscrew the ocular from the dermatoscope and remove the rubber ring
- ◆ Put the spacer (2) around the ocular
- ◆ Slip the camera with adapters onto the dermatoscope (click fastener)

Camera settings :

Attention: Apply only when using the DermoGenius software:

Please make sure, that a SD memory card is always installed in the camera, as the image transfer from camera to the DermoGenius database is not functioning automatically out of the internal memory of the camera.

The correct camera settings are done by our staff when the camera is delivered. In case the settings have to be done later on again, please choose the following settings (see also camera manual, model Rollei compactline 130):

1. Switch camera on
2. Switch off flash, set focus mode to "Macro"
3. Call up menu
4. Choose in the menu „Resolution“ the following settings:
Picture size: **5** or **8 Megapixel** (recommended)
5. Choose in the menu
 - „Quality“ >> Fine
 - „Metering“ >> Multi
 - „ISO“ >> Auto
 - „White Balance“ >> Daylight
 - „Exposure“ >> 0 EV
 - „Drive Mode“ >> Off
 - „AEB“ >> Off
 - „Highlight“ >> Off
 - „Sharpness“ >> normal
 - „Effect“ >> Off
 - „Date stamp“ >> Off
 - „Stabilizer“ >> Off
 - „Face tracking“ >> Off
 - „Foto frame“ >> Off
 - „Digital Zoom“ >> Off